EN

- A. FERTI·LILY CONCEPTION GEL
- B. HOW TO USE THE FERTI·LILY CONCEPTION GEL
- C. PRECAUTIONARY WARNINGS
- D. INGREDIENTS

A. FERTI·LILY CONCEPTION GEL

The FERTI·LILY Conception Gel is a personal lubricant that can be used while trying to conceive. The Conception Gel is clinically proven to be friendly to sperm. The FERTI·LILY Conception Gel is isotonic, has a pH friendly to sperm survival and mimics the properties of the fertile fluids. The FERTI·LILY Conception Gel can be used in combination with the FERTI·LILY Conception Cup to increase your chances of conception.

B. HOW TO USE THE FERTI·LILY CONCEPTION GEL

Step 1
Remove the seal from the tube before first use.

Step 2
Apply the lubricant to the outside of the vagina or to the shaft of the penis prior to intercourse.

Squeeze a small amount of product onto your fingers and/or the palm of your hand and apply to genital area. Vary amount to achieve desired lubrication.

Step 3
After each use, firmly close the tube. Do not leave the tube open between uses.

C. PRECAUTIONARY WARNINGS

If you have sensitive skin, try applying a little gel on the inside of your elbow to test if you are sensitive to its ingredients. The FERTI·LILY

Conception Gel can be used for 3 months after opening. Do not use the FERTI·LILY Conception Gel if you have a yeast or bacterial infection or if you have recently had irritations or wounds on the vaginal area. Resume use once the infection has completely cleared. If you experience any adverse effects, discontinue use and consult your physician. Keep out of the reach of children.

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